

Cleaning & Sterilisation Instructions For: Mouth Props

Manual:

- 1. Pre-disinfect proprietary product follow the manufacturer’s instructions.
- 2. Brush under running water - wearing suitable P.P.E.
- 3. Dry thoroughly.

Mechanical:

- 1. Place in the washer/disinfector.
- 2. Use a pH neutral detergent according to the manufacturer’s instructions.
- 3. Autoclave at 134°-137°C for 3-3.35 minutes.
- 4. Dry thoroughly with a lint free cloth.



REF	SRU010 (116496UNODENT)	REF	SRU030 (116498UNODENT)
REF	SRU020 (116497UNODENT)	REF	SRU040 (116499UNODENT)

 Young Innovations, Inc. 2260 Wendt Street Algonquin, IL 60102
www.younginnovations.com (800) 325-1881

 EU REP Young Microbrush Ireland Ltd. Clogherane, Dungarven
Co. Waterford, X35 VE02, Ireland

UK ResponsiblePerson Astek Innovations Ltd. 95 Atlantic Street,
Altrincham WA14 5DH United Kingdom Tel: +44 (0)161 942 3900

 DD Products & Services Ltd. 6 Perry Way, Witham, Essex, CM8 3SX.
Tel: (01376) 500582. Email: feedback@unodent.com

 Trasmore Ltd, T/A DMI, Kellystown House, Kellystown Lane, Leixlip,
Co Kildare, W23 VF9P, Ireland Tel: +353 1 4273700

Symbol	Description	Symbol	Description
REF	Product reference number	MD	Medical Device
134°C	Sterilizable in a steam sterilizer (autoclave) at temperature specified		Sterilizable in a steam sterilizer (autoclave) at temperature specified
CC	Country of Origin		Importer
UK CA	UK Conformity Assessment	EU REP	Authorized representative in the European Community
NON STERILE	No Sterile		Distributor
	Do not use if package is damaged	CE	CE marking Conformity
	Consult instructions for use	LOT	Batch or lot code
UDI	Unique Device Identifier		Use By Date